



WHILL Model C2

Adventure awaits. Where will you go?



Adventure awaits. Where will you go?

Designed for the world you live in, the WHILL Model C2 brings you there with the confidence you deserve.

It's time to challenge the perception of what a mobility device can be.



Portable

Disassembles in four easy steps.

The WHILL Model C2 can be disassembled for easy transportation in the trunk of any car. Whether you're shopping on the weekend or traveling with your family, the WHILL Model C2 allows you to live life to the fullest.

Seat Assembly
26.9 lbs.

Front Drive Base
38.8 lbs.

Rear Drive Base
42.8 lbs.

Battery
6.0 lbs.



Remove the battery.



Raise the lever under the seat.



Pull the seat upwards.




Pull the lever on the side of the battery toward you to separate the parts.


*Model C2 is not to be used as a seat while it is being transported.

The information contained herein is correct at the time of printing, and we reserve the right to alter specifications without prior notice.

Features and Functions

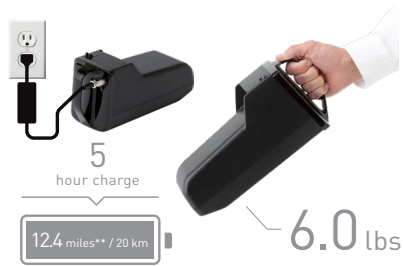

The battery can be removed and charged in a regular household outlet.

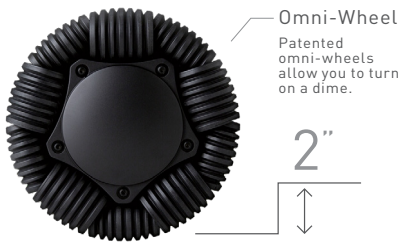

Effortlessly climb over obstacles up to 2" in height with two powerful motors and large front omni-wheels.



Drive remotely, lock/unlock, and manage speed settings via the WHILL



The WHILL Model C2 can make tight turns, allowing you to go places that a typical scooter can't take you.

The controller can be positioned on the left or right side.



Drive straight on a tilted path.

****Note:** Driving range of 12.4 miles will vary due to user weight, drive surface, terrain, ambient temperature, speed setting, and battery charge and condition.

Customization

Choose from 6 different arm color options and customize the WHILL Model C2 to match your personal style.

Color Options



- White
- Black
- Light Green
- Red
- Blue
- Pink



Accessories

◊ Non-medical options



Smart Key



Joystick



Walking Stick Holder



Adjustable Foot Support



Waterproof Cover◊



Controller Cover◊



Arm Pads◊



Side Bag◊

Specifications

Driving Range	Up to 12.4 Miles	Maximum Speed**	5 MPH
Max Incline	10°	Charge Time	~ 5 hours
Obstacle Clearance	2"	Weight Capacity	Up to 300 lbs
Turning Radius	29.9"	Suspension	4 Wheel Independent
Device Length	38.8"	App Enabled	Yes, with remote control
Seat Sizes	16" /18" /20"	Swappable Arm Covers	6 colors
*Overall Width	21.8" /23.6" /25.6"	Battery	Li-ion/6 lbs
Weight	114.4 /116 /116.4 LB	Air Travel Approved	Yes
Controller Height	26.4" /27.2" /28.0" /28.7"	Water Resistant	IPX4
Seat Height	15.9" - 19.1"		
Back Support Angle	90° /95° /100°		

**Acceleration and/or deceleration may change when driving on steep slopes.

WHILL Inc.

303 Twin Dolphin Dr., 6th Floor, Redwood City, CA 94065
(844) 699-4455 us.info@whill.inc www.whill.inc



Indications for Use: The intended use of the WHILL Model C2 powered wheelchair is to provide outdoor and indoor mobility to persons limited to a seated position that are capable of operating a powered wheelchair.

Wheelchair Class: A FDA Compliance: The WHILL Model C2 has received 510(k) clearance for marketing as a Class II medical device.

740-00235